


SECURE LOOPS™

**A NEW PRODUCT DESIGNED TO FACILITATE QUICKER -
EASIER WHEELCHAIR TRANSPORTATION SECUREMENT**

SECURE LOOPS™ Manufactured of high strength "TENDER STRAP"™ webbing are uniquely assembled into adjustable semi-permanent loops to provide easily accessible "Flex" rings for wheel chair tie-down strap attachment regardless of the tie-down system used.


IMPORTANT: SECURE LOOPS Must be attached to the strongest support structure on the wheelchair. Their location may vary from the four corners of the seat structure on a standard wheelchair or to the supporting structure underneath the seat and steering wheel pedestal base of a 3 wheeled scooter.

CAUTION: Never attach **SECURE LOOPS** to wheelchair wheels (in most cases, these are the weakest tie-down locations).

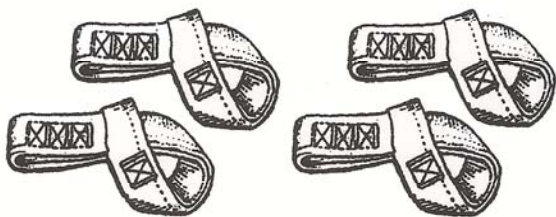

INSTALLATION:

1. Open the Secure Loop to its flat length position.
2. Encircle wheelchair tubing structure (or frame) large loop forward. Insert large loop into small loop.
3. Push large loop end through small loop until sewn end of large loop 'locks' small loop into place.
4. Attach each wheelchair tie down strap by inserting the snap hook end fitting through the webbing loop you have formed, then snap hook to metal ring end fitting locked securely into floor track.

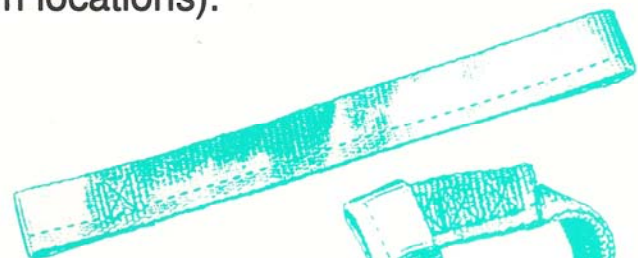

1



2



3



4